

Setting Parameters

Dental Wings for cara Print 4.0

Important information:

- Observe the Instructions for use of the material.
- The setting parameters listed below are for dima Print resins determined with cara Print 4.0
- The given dimensions serve as guidelines and may be customized according to individual wishes.



Software Setting Options:

As of 07.2019

dima Print Stone beige	
Pin Shape	Standard
1. Margin Expansion	0.20 mm
2. Fit Distance	0.06 mm
3. Ditch Height	1.00 mm
4. Ditch Depth	0.20 mm
5. Stump Conic Height	4.00 mm
6. Stump Slope	2.00°
7. Pin Diameter	3.00 mm
8. Pin Emboss Height	1.00 mm
9. Pin Emboss Offset	2.00 mm
10. Pin Fit	0.06 mm



Contact in Germany

Kulzer GmbH
 Leipziger Straße 2
 63450 Hanau, Germany
cara-service@kulzer-dental.com
kulzer.com/cara

AL108 EN 07/2019 ORT

Giving a hand to oral health.



KULZER
 MITSUI CHEMICALS GROUP